

Vital-Cath™ 72 Plus

Closed Suction System

Specifications

Product Code	Outer Diameter	Remark	Catheter Length (mm)	Packaging	ISO Colour
6351000A / 6351000AM	10Fr / 3.33mm	Endotracheal	540	20ea/box	●
6371000A / 6371000AM		Tracheostomy	305		
6381000A / 6381000AM					
6351200 / 6351200M	12Fr / 4.0mm	Endotracheal	540	20ea/box	○
6371200 / 6371200M		Tracheostomy	305		
6381200 / 6381200M					
6351400 / 6351400M	14Fr / 4.67mm	Endotracheal	540	20ea/box	●
6371400 / 6371400M		Tracheostomy	305		
6381400 / 6381400M					
6351600 / 6351600M	16Fr / 5.3mm	Endotracheal	540	20ea/box	●
6371600 / 6371600M		Tracheostomy	305		
6381600 / 6381600M					



- M = with MDI port
- All adult size Vital-Cath™ package includes disconnect wedge and corrugated adapter.



① Irrigation Port

One way valve prevents backflow leakage during cleaning and aerosolized secretion contamination.

④ Better Cleaning

When the isolated valve is on OFF position, the irrigation chamber is 100% sealed allowing better cleaning results and prevent the back flow of irrigated saline into patient's trachea.

② Operation for Isolation

- Suction with simultaneous ventilation is available at **ON** position.
- At **OFF** position continuous ventilation is provided for patient comfort.

③ MDI Port (optional)

Large easy-grip finger contact area allows convenient and fast medicine introduction.

⑤ Double-swivel elbow

Reducing strain on ventilator tubing for patient comfort.

⑦ Black Ring Indicator

Always make sure to withdraw the suction tube completely until the black ring indicator is visible before turning the valve to "OFF" position.

⑥ Catheter Tip

The black safety ring at the tip of catheter provides clear indication when suction catheter is damaged due to incorrect usage.

Additionally, with the withdrawal of suction catheter, the black ring is able to indicate whether catheter has entered irrigation chamber.

**Better Design,
Better Patient Comfort**